

# AutoScope

Customized product to score Terasaki plates on Microscopes



## “AutoScope” features:

- Secure handling of Terasaki plates on different microscopes.  
→ **We also offer suitable Terasaki plates.** Please contact us for more information.
- Automatically advances Terasaki plate on your microscope as the scores for each well are entered.
- Stage movement to any well of the plate with one click of the mouse.
- Stage is controlled by the SystemLink HistoScope software (see below).
- Stages are available for CK2, IX2-SP, and IMT-2 microscope from Olympus and Ts2R from Nikon Eclipse.
- Customized stages for other microscopes are possible.

## “HistoScope” features:


- Microsoft Windows-based software operating the automated microscope stage in clinical Histocompatibility laboratories.
- Microscope workstation can be networked to review scores at your desk.
- Two software versions for SQL-servers or local computers (Access-based) are available.
- Patient and donor database with search tools included.
- Efficient movement of the motorized stage, using the controller keypad or the computer keyboard.
- Score, re-score and double-score features available.
- Archive and restore functions for trays and scores in the software.
- Patient sample search tool enables easy loading of screening crossmatch trays.
- Import/Export function allows sharing tray definitions with other HistoScope software users.

### Computer requirements to install “HistoScope” software:

- PC with Pentium Processor
- Microsoft® Windows 7+
- 128 MB memory or more
- VGA monitor (1024 x 768)
- Server installed with Microsoft® SQL Server 2012+ or Microsoft Access 2010+ (stand-alone only)

### Installation of the “AutoScope”:

1. Install customized stage on your microscope
2. Connect motor cables to power cable
3. Connect power cable to control box
4. Connect power cord to control box
5. Plug communication cable into COM port on computer
6. Plug power cable into power receptacle (not in the image)
7. Turn on power switch
8. Download “HistoScope” software via download link
9. Arrange date for software instruction through video conference

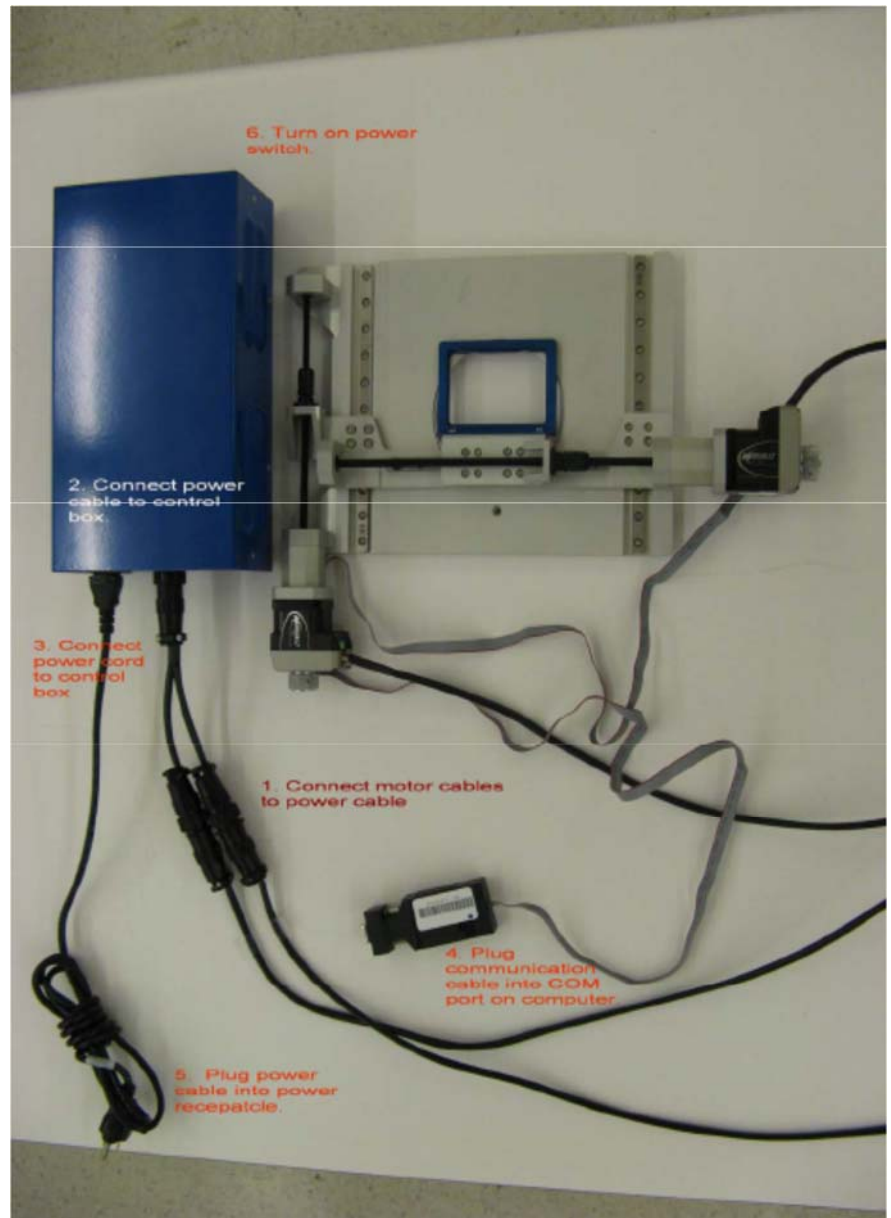

➔ **Please do not hesitate to contact Dunn Labortechnik if you should need any support with the installation.**